



## MAGIC ACUPOINT PROBE

### **Contraindication and points for attention**

1. Do read the instruction of points for attention and manipulations carefully before use.
2. Do take the battery out of the product if it is not going to be used in a long time .In case the liquid of the battery leaks out and damages the product.
3. Do not dismantle or repair the laser cure instrument in person, or it will lead to incidents or break down.
4. Do put the product in the place which is untouched for children.
5. Do stop using the product if any trouble is caused by the treatment and consult the doctor in time.
6. Do not put the product in the environment of high temperature, damp place. Do not fiercely press and collide it as it is a precise electrical product.
7. Do keep hands clean and by dry when use.
8. Do not check when the air temperature is high or low. Do not check when you are full or hungry .Do not check after drinking wine, working or doing sports. In case the result is not correct. Check when you are calm and the arid temperature is not too high or too low .Do not excessively wipe and wash ear and do not strain ear before checking in case the ear become bloodshot, therefore after ear's resistance and have the result of false masculine.
9. Do use ethyl alcohol to clean the probe before and after checking .If parts of the ear have inflammation and are flare, do not touch them.

### **Specifications**

1. Frequency: 105 HZ Auto change
2. Length of pulsate : 300~500  $\mu$  S Auto change
3. Output : < 120V<sub>p</sub>-<sub>p</sub>(500  $\Omega$ )
4. Power source: Battery # 7 (AAA) 4

**Elimination of common breakdowns**

Breakdowns	Possible cause	Solutions
Indicators have no response to the pressing of the buttons for power source	Put power source or battery in wrong position, not touching the conductor with finger	Put power source into checking button, touching the conductor with finger, check battery
No prickle feeling during treatment	Put power source in wrong position, electric roll is not hold inside the hand during treatment	Put power source into treatment button, electric roll is hold inside the hand during treatment
Checking indicator are darker than usual	The volume of batteries is not enough	Renew batteries

**History of Ear Acupoints Treatment**

Ear acupoints treatment is an important part of Chinese Acupuncture and Moxibustion. It diagnoses, prevents and cures diseases and keep health care by means of auricle. If there is something wrong in the body, there will be reaction points on the auricle and the stimulation could be offered to the points in order to cure diseases.

Ancient medical experts had accumulated a lot of experiences about the relationship between ear-acupoints and the whole body. 2500 years ago, Canon of Medicine recorded more than 30 examples of ear acupoints applications for curing diseases. Chinese ancient books first recorded more than 10 ear acupoints and declared that auricle's belonging to five organs. Picture of auricle was also depicted. (See Fig.1) Afterwards, P.Nogier from France spent 6 years on ear acupoints research, and declared that there's corresponding relationship between the auricle and the whole body. He had found 42 ear acupoints by 1957 and published the systematic figure of ear acupoints. It was brought into China in 1958 and was developed from 42 to more than 200. Now there are over 90 countries and regions using Ear acupoints Treatment due to the wide spread of it during 30 years. In order to meet the international exchange need, WHO appointed China as producer of draft for the International Standard of Ear Acupoints Plan in 1982. The Ear Acupoints Corporation Team is formally set up. Through clinical research over these years, 91 ear acupoints are approved world wide. The Ear-acupoints treatment consisted of many kinds such as Ear Acupuncture and Moxibustion, Laser treatment, Photo-electron treatment etc.

**Curative principles**

Energy channels of the ear are complicated and they are the energy channels of body meet. They communicate with all the energy channels from inside to outside, from upper part to lower part. When body or organs have diseases, the earpoints will appear electrical changes (which called respondent dot of masculine). According to this principle, the Earpoints probe can find the respondent dot of masculine. Light to show the condition of body. It is the auxiliary machine of diagnosing and consulting; what's more, it helps the cure of pasting and pressing ear acupuncture point to improve curative effect. After many practices ,cure disease at ear acupuncture point can mediate energy channels, can regulate the function of vigor or the body organs ,strengthen the ability of resisting disease, accelerate the circulation of blood ,reach the function of preventing disease., and caring your health.

**Characteristics:**

Auricle is the place where the channels come to together. And when people is ill, the reflexion region will be led to some changes, called positive reflection point. This instrument can find the positive reflection point correctly and show the body condition by lighting different lights. It can also used in


earpoint therapy and bodypoint therapy. So it is the combination of earpoint therapy and assist diagnosis.

- 1.The lamp lights up as soon as you press the button. It can point out the positive reflection point correctly and cure disease without the help needles.
- 2.The instrument is used in two ways and is convenient to family members.
- 3.It shows our body condition by lighting different lamps.
- 4.The high rate of finding positive reflection point will allow us to cure the disease effectively.
- 5.Multiple uses: earpoint checking, earpoint therapy and bodypoint therapy.
- 6.Wide cure range.
- 7.Easy to carry. Power from 4 granules AAA

### Instruction of all parts of the product

(1) Before adjusting the datum point

Put power source(6)left side into checking button, user can hold magic Acupoint Probe& thumb press on the conductor(4), forefinger adjust the checking wheel (2), into Zero(0), then start adjusting the datum point.

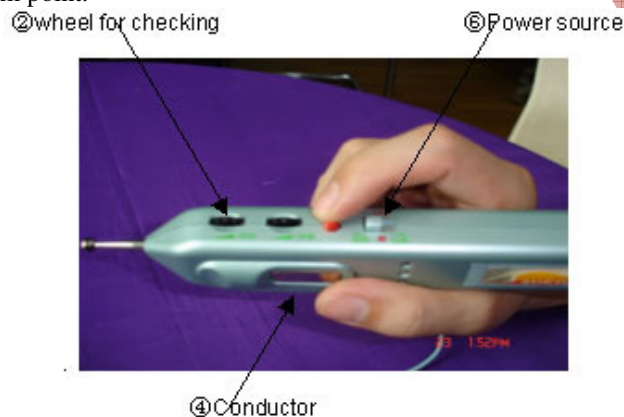

(2) Adjusting the datum point

Press the probe on the ear root and then adjust.  
PS:

Adjust before each checking

Do clean the probe by ethyl alcohol after checking each person

If there are scars cause by inflammation at earpoints, do not touch these parts.

First ,use forefinger to push thechecking ware forward to the end. And then slowly adjust until the green and the yellow lights go out, and the red one is bright. That is the standard dot.Stop when the red light is just bright and hearing the sound of 'mi mi', do not adjust too much.

(PS: (1) slowly wriggle the button ,do not wriggle quickly;


(2) after adjusting ,do not touch the push-button when checking)

(3)Result of survey

One hand slightly pull the ear, the other hand hold the probe to check. Do hold horizontal of 90 degree checking, enough pressure not too much, press to acupuncture point when the light is bright.



Green light indicates negative Reaction (good)

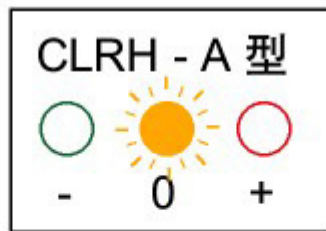

Yellow light indicates ok




Red light indicates positive reaction (bad) (if positive reaction, the machine will sound 'mi mi')

**Suggestion of checking steps:**

Begin from the helix:

- 1) the helix
- 2) the seaphoid fossa
- 3) the body of the hedix
- 4) the triangular fossa
- 5) the superior concha
- 6) the inferior concha
- 7) the antitragus
- 8) the lode
- 9) the tragus

**Close the instrument**

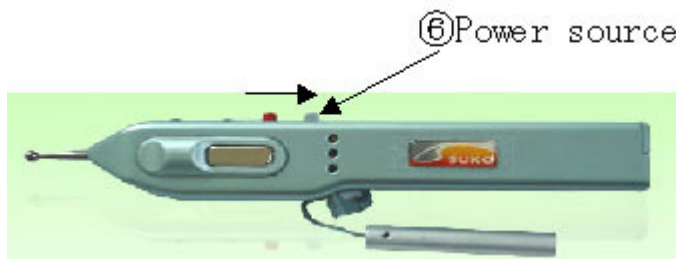

**Matters needing attention when checking:**

- 5) Do not massage ear too much before checking ,in case the ear become bloodshot ,having false masculine.
- 6) Do not check when you are excited or in bad mood, the air temperature is too high or too low and you are full or hungry. Do not chekck after drinking ,working or doing sports. These will affect the result of survey. Do check when you are calm and the air temperature is not too high or too low.
- 7) Do adjust the standard dot precise, too high will bring false masculine, and too low will lose information.
- 8) When checking the probe must be vertical with th skin,the pressure of probe must well-distributed, the pressure is the same as you find the standard dot .Press the era acupuncture point in order, find out the reacting dots of masculine. (They are the place when the red light is bright or you feel pain when pressing.)


Preventing confuse pathological reaction dots and physiological electric good conduction .For example, when checking Internal Genital of the Triangular Fossa,Angle of Superior Concha ,Endocrine of the incisura Intertragica, the Seaphoid Fossa and so on,which are physiological electric

good conduction ,the red light may be bright ,but not meaning there are pathological reaction dots. The pathological dots arde always tenderness, so tenderness should be checked when the red light bright, if there is tender on the point, which may be reaction dot, suggesting that the organ that the earpoint relates to may have disease. So treatment should be put on this point, such as irradiating with laser rays, acupuncture, and magnetic beads pressing. However, when treating on earpoints, all them may not have evident tender, so the points on which the red light is bright (with alarm) should be chosen to be cured.

#### Instruction of Ear/ Body Treatment


##### (1) Step of treatment

##### Choose treatment stick before treatment

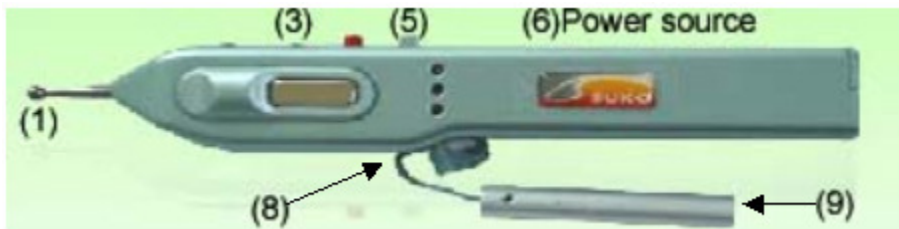

After find out the treatment acupoint ,put power source (6) button into pulsate treatment button.

##### (2) Adjust volume of pulsate

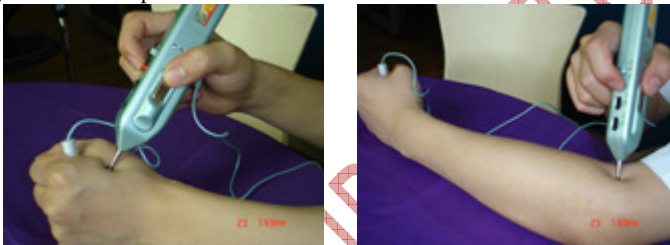

**User should hold electric roll in right handside when right body acupoint during treatment.**

##### (3) Attention during treatment

- 1) Don't make the pulsate treatment too long in each acupoint & avoid acupoint of near face & heart.
- 2) Please let user know that it should be pain & swelling during treatment on acupoint
- 3) Don't consult the doctor before using the instrument if you are under one of the following situations :Neoplastic patients; Pregnant women; Zymotic patients; Acute diseases' patient;
- 4) Cannot make the machine wet , otherwise it will damage.
- 5) User should hold electric roll in left handside when left body during treatment.
- 6) Don't use it carbuncular position.